USER MANUAL

Relaxation splint for grinding

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00022113 M: 22113

EN

Ladies and Gentlemen, thank you for purchasing our product!
Before starting to use, please read the following instructions
for the correct use of the product. Please keep these instructions on
future and to follow its recommendations, as non-compliance may endanger life or health.

Important: KEEP FOR FUTURE REFERENCE!

- Please read it completely before use and follow all instructions in the manual.
- Do not use the product if any part is missing or damaged.
- Please keep this manual for future reference.

SPECIFICATION:

- Dimensions: 6.5x7cm

Rail preparation method:

boiled water should be poured into a glass container. When the water cools down to approx. 70 ° C, insert the rail into the container for 20-30 seconds. When the vertical walls of the rail slightly deviate to the horizontal, the rail should be placed over the jaw by biting it. After 1-2 minutes, remove it from your mouth and place it in cold water for 2-3 minutes. The rail is ready for use (if the rail is too big, it can be cut).

CHARACTERISTIC:

The relaxation splint is a way to relieve the symptoms of clenching and grinding teeth. When this happens, a person clenches the jaw up to 10 times harder than when, for example, eating.

ATTENTION:

- Do not wash the splint in hot or boiling water
- The slightly cracked rail is suitable for further use
- Stop using the rail if it is deformed and/or completely cracked

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