soulima.[®]

USER MANUAL

Bunion corrector

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Opaska żelowa na haluksy

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Gelový pásek na boule

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INSTRUCCIONES DE OPERACION

Banda de gel para juanetes

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Fascia in gel per borsiti

MANUAL DE UTILIZARE

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Gélpánt bűtykökhöz

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Gelový náramok na vbočené palice

KASUTUSJUHEND

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Gēla lente bunjoniem

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00010814

EN

Ladies and Gentlemen, thank you for purchasing our product!
Before starting to use, please read the following instructions
for the correct use of the product. Please keep these instructions on
future and to follow its recommendations, as non-compliance may endanger life or health.

Important: KEEP FOR FUTURE REFERENCE!

- Please read it completely before use and follow all instructions in the manual.
- Do not use the product if any part is missing or damaged.
- Please keep this manual for future reference.

SPECIFICATION:

- beige colour
- set: pair;
- material: elastic,
- dimensions (width/height): 7x9cm;

CHARACTERISTIC:

The band restores the correct position of the fingers and prevents them from overlapping, which later helps straighten and align crooked fingers. The cover perfectly protects the bone against side impacts and micro-injuries. By isolating the painful area, it accelerates skin healing and reduces pain associated with pressure and friction. The ergonomic, anatomically adjusted shape means that the band can be used by both women and men. The universal size makes the cover comfortable, even if worn all day long. The band is reusable and can be worn both under socks and with exposed shoes. The very soft and flexible gel perfectly absorbs impacts, while preventing the formation of corns, sores and abrasions. The gel is also an anti-allergic product that is easy to keep clean.

SAFETY RULES:

- Start using concealer gradually. At first, wear it for a short time (e.g. 1-2 hours a day) and gradually increase the wearing time to avoid irritation and discomfort.
- Regularly check the condition of the skin in places where the corrector comes into contact with the foot. Avoid using concealer if you notice any abrasions, redness or blisters.
- Keep your concealer clean to prevent the growth of bacteria and fungi. Wash your concealer regularly according to the manufacturer's instructions.
- Wear appropriate footwear that is comfortable and does not put pressure on your feet. Shoes should have a wide toe to provide adequate space for the toes.
- If you experience pain, discomfort or notice deterioration of your feet while using the corrector, stop using it immediately.

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